



ProGienics

For Proactive Hygiene

Surgical Instrument Multi-enzymatic Cleaner

FOR CLEANING

*Surgical Instruments, Medical Devices and
Sterile Surfaces before disinfection/sterilization*

ProGienics Surgical Instrument Multi-enzymatic Cleaner is formulated to remove proteinaceous matter from surgical instruments and medical devices. For use as a presoak, manual cleaner, or in ultrasonic cleaning equipment and washer sterilizer equipment. Designed to clean surgical equipment, even the most difficult to clean. Effectively removes blood, tissue, mucous and other proteinaceous soils, as well as fatty tissue. Medical devices should be sterilized or disinfected by appropriate means in a separate subsequent step. Will not stain instruments or equipment made of glass, metal, plastic, or rubber.

DIRECTIONS

For presoak, add on (1) ounce or 30 mL of PROGIENICS ENZYMATIC CLEANER & PRESOAK to one (1) gallon or 3.79 L of water. For best results use water temperature at 77-130 F (25-55 C). Do not exceed 140 F (60 C). Soak according to facility protocol. After soaking, rinse thoroughly with water and transfer to the next cleaning operation such as manual ultrasonic, or washer sterilizer cleaning. Discard used solutions daily or when visibly soiled. Do not add any other chemicals to the product, such as bleach, disinfectants or detergents. Added chemicals may destroy the effectiveness of this product.

For Surface Cleaning: Dilute 2 to 4 ounces per gallon of water. Spray the surface from a distance of 6-8 inches. For heavy soil use a synthetic brush to agitate the surface. Remove soil using a damp sponge or lint-free cloth. Rinse thoroughly with water and disinfect according to the instructions on the product label. For general surface cleaning, spray the solution evenly across the surface and wipe clean using a sponge, lint-free cloth or paper towel. Rinse the surface thoroughly with water and disinfect according to instructions on product label.

PHYSICAL AND CHEMICAL PROPERTIES: An enzymatic detergent that is completely soluble in water. For professional use only. Not for home use. See Safety Data Sheet for additional information.

TYPICAL PROPERTIES: pH: 7.5 – Solids: 11.5 – Appearance: Clear green liquid – Fragrance: Pleasant

SAFETY: WARNING: Keep out of reach of children. Read label and SDS before use. Causes serious eye irritation. Flammable liquid and vapor.

PRE-SOAK

SURGICAL INSTRUMENTS & MEDICAL DEVICES



- Cannulated instruments
- Biopsy forceps
- Flexible forceps
- Flexible Fiber Optic Endoscopes
- Rigid scopes
- Manual Resuscitators & Apparatus

PRE-CLEAN

ANIMAL CARE EQUIPMENT



- Animal cages
- Animal enrichment
- Food Preparation Equipment
- Bedding dispensers
- Instrument stands & Trays
- Wet tables

PRE-SCRUB

NECROPSY EQUIPMENT



- Trimming Tables & Stations
- Scales
- Lift tables
- Transport Carts & Tables

